



SuperTabs-A

Powerful Autoclave rinsing tabs

Safe • Easy • Effective

Well-functioning autoclave with the new Forumtec SuperTabs-A.

Safety - Condensed tabs with no risk of hazardous fluids or powder

Super effectiveness - Handling corrosion and sediments for high efficiency

Value - Proper functioning and durability of the autoclave. For S and N class autoclaves



Secure your autoclave maintenance



SuperTabs-A

Autoclave cleaning tablets.

Instructions for use:

Cleaning and descaling agent designed specifically for cleaning and removal of water deposit oxides and other sediments appearing in steam sterilizers.

The active ingredients are a combination of acidic salts and other cleaning agents.

Use 1 tablet to clean your Autoclave.

Clean every 20 cycles or once a week.

Suitable for class S and N autoclaves.

Cleaning procedure:

1. Remove all instruments and trays from the chamber.
2. Remove the water from the reservoir. Fill it with distilled water.
3. Place one tablet inside a cold chamber.
4. Run one unwrapped cycle, select 30-minute sterilization time at 134°C - 137°C, no need to dry.
5. Remove the water at the end of the cycle including the reservoir.
6. Refill the water reservoir with distilled water.
7. Run new unwrapped cycle, without a tablet, select 30-minute sterilization time at 134°C - 137°C.
8. Drain again the water including the reservoir.
9. Switch off the autoclave and wait until the chamber will be cold.
10. Wipe the chamber. Be careful, make sure the chamber is cold!
11. Drain again the water from the reservoir.
12. Refill with distilled water only.
13. The autoclave is now clean and ready to use.

IMPORTANT: Do not sterilize instruments during the cleaning process!

Caution: Wearing gloves is recommended when handling the tablets.

Contains mildly acidic ingredients. Avoid contact with the skin, eyes or clothing.

Wash hands well after touching the tablets. In case of eye contact, flush with running water for at least 15 minutes. If irritation persists, get medical attention.

If the tablet is accidentally swallowed, do not induce vomiting. Drink large amounts of water and obtain medical attention.

Keep out of the reach of children!