



## SuperTabs-U

**Dual-action Ultrasonic rinsing economical tabs**  
**Cost-effective • Efficient • Powerful**

Forumtec presents the first Ultrasonic-cleaner cleaning tabs.

**Economy** - the condensed tabs are cost-effective over other available solutions

**Easy to use** - eliminates the need of handling heavy containers

**Dual-action mode** - a powerful combination of Oxygen bleaching effect and Enzymatic activity for ultimate outcome



Economize your Ultrasonic cleaning



**SuperTabs-U**

## Ultrasonic cleaner tablets.

### Instructions for use

1. One tablet makes 3-liter solution.  
Institutional size tablets are available as well.
2. Do not mix with any other detergents or disinfectant products.
3. Drop one tablet per every 3 liters of water directly into the tank.
4. Fill the tank with warm tap water, 40°C - 50°C. The water temperature should not exceed 60°C.
5. Ultrasonically clean the items for a minimum of 10 minutes.
6. Instruments should not be soaked in the solution for more than 24 hours.
7. For ultrasonic cleaners follow equipment manufacturer's recommendations.
8. Dry the instruments before proceeding with the sterilization process.
9. Replace the solution when it becomes visibly cloudy, or at least once a day.

**Caution:** Gloves recommended when handling the tablets. Avoid contact with the skin, eyes or clothing.

Wash hands well after touching the tablets. In case of eye contact, flush with running water for at least 15 minutes. If irritation persists, get medical attention.

If the tablet is accidentally swallowed, do not induce vomiting, drink large amounts of water and obtain medical attention.

Keep out of the reach of children!

