

Ultrasonic cleaner tablets.

Instructions for use

1. One tablet makes 3-liter solution.
Institutional size tablets are available as well.
2. Do not mix with any other detergents or disinfectant products.
3. Drop one tablets per every 3-liters directly into the tank.
4. Fill the tank with warm tap water, 40°C-50°C. The water temperature should not exceed 60°C.
5. Ultrasonically clean items for a minimum of 10 minutes.
6. Instruments should not be soaked in the solution for more than 24 hours.
7. For ultrasonic cleaners follow equipment manufacturer's recommendations.
8. Dry the instruments before proceeding with the sterilization process.
9. Replace the solution when it becomes visibly cloudy, or at least, once a day.

Caution: Gloves recommended when handling the tablets. Avoid contact with the skin, eyes or clothing.

Wash hands well after touching the tablets. In case of eye contact, flush with running water for at least 15 minutes. If irritation persists, get medical attention.

If the tablet is accidentally swallowed, do not induce vomiting, drink large amounts of water and obtain medical attention.

Keep out of the reach of children!

